

FOREARM WINGLET FOAM SET INSTRUCTIONS



Purpose:

Both the Midi and Maxicuff circumference are identical, so to reduce the diameter of the forearm support cuff foreither forearm winglets maybe inserted. This maybe necessary for people with narrower forearms who want a more snug fit.



Warning!

Exchanges and returns will not be accepted once the wing tips have been permanently adhered to the plastic. Please first set the mup and check they fit before doing this.

- 1 First try the smartCRUTCH for size (bear in mind, bulky Winter clothing could be a factor.)
- 2 Test-fit the first piece as shown in Fig.1 with the white adhesive cover placed on the inside of the cuff and the winglet foam aligned with the existing foam.
- 3 Once you are satisfied, remove the white adhesive cover (Fig.2) and stick in place.
- 4 Repeat with the remaining 3 pieces.

